



SPEEDY TEXTUR & FITTING KIT

- All-purpose surface texture- and accurate fitting cream-

Put an end to dirty fingers, models and working environment while working with surface texture, silver powder, occlusal sprays, etc. The innovative *SPEEDY TEXTUR & FITTING KIT* makes this possible.

Application surface texture:

The *TEXTUR- & FITTING-CREAM* is thinly applied directly on the model/crown with the *TELE-TEXTUR- & FITTING-BRUSH* with its practical aluminium protective covering. The surface texture, the shape and the overall harmony of the work are therefore visually highlighted without distracting colour differences. It is straightforward to be examined if the structures and transitions are soft and fluent. Nuances are clearly visible and easily corrected.

Application of the accurate fitting cream:

The *TEXTUR- & FITTING-CREAM* is especially appropriate for the visualisation of imperfections when fitting crowns and secondary components, when fitting basal surfaces for pontics, when grinding in approximal contact points and when systematically grinding in occlusion. The *TEXTUR- & FITTING-CREAM* is an all-purpose cream for metal, full ceramics, zirconia, acetal, Polyapress and for all synthetic materials. The colour application marks repeatedly. Important: Apply *TEXTUR- & FITTING-CREAM* **very thinly!**

Please observe:

- Move the tip of your brush very lightly across the *TEXTUR- & FITTING-CREAM* and apply it thinly
- Clean the tip of your brush regularly with a tissue
- When not in use, keep your brush in the telescoping aluminium protective covering
- Always keep the container shut properly
- The *TEXTUR- & FITTING-CREAM* can easily be removed through steam

Tips:

- Look at your work with the *TEXTUR- & FITTING-CREAM* with light coming from the front and side. This way the various outlines of the structure can be better seen.
- For the final touches of the shape correction. The shape correction can easily be examined with the flat mirror, which once opened is simple to handle. Look at your work from the perspective of your patient. You will have a total new approach to your work. The slightest shape disharmony can be recognized immediately and accurately corrected. Your favourite perspective is then out of action.



ORDER

simply by FAX 0821/5892553

Description	Article no.	Contents	Unit price*	Amount	Total price*
SPEEDY TEXTUR & FITTING KIT	A71004	7ml tin Textur- & Fitting-Cream, silver 1 Tele-Textur- & Fitting-Brush, red marten hair 1 Flat-Mirror	26,50		
Textur- & Fitting-Cream	A71005	7 ml tin Textur- & Fitting-Cream, silver	13,40		
Tele-Textur- & Fitting-Brush	A71006	1 Tele-Textur- & Fitting-Brush, red marten hair	12,90		
Flat-Mirror	A71007	1 Flat-Mirror	5,50		

*Prices do not include VAT and Postage

Sender:

Forename _____
 Name _____
 Company _____
 Street + No. _____
 Town + Postcode/Zip Code _____
 Date, Signature, company stamp _____

WICHNALEK
 Dentale Schulung/Entwicklung/Vertrieb
 ZTM Norbert Wichnalek
 Hochfeldstr. 62
 D-86159 Augsburg
Or simply by Fax
+49(0)821/5892553
 Fon +49(0)821/571212